

# PORTLAND ELASTIC NAIL

Six titanium alloy nail diameters for all indications

Nail tip allows easy nail insertion and sliding along the medullary canal

Two sizes of sharp self-cutting thread end caps to cover all nail diameters, Provide additional axial stability in unstable situation

Allows earlier weight bearing, minimal skin incision is some of advantages of the elastic nail.


for 1.50 - 2.50 mm




for 3.00 - 4.00 mm



Ø 1,5 mm

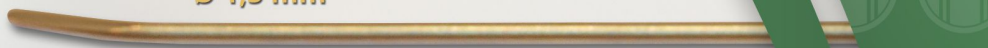

Ø 2 mm

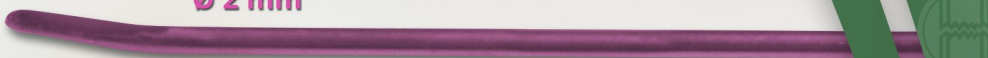

Ø 2,5 mm



Ø 3 mm



Ø 3,5 mm



Ø 4 mm





#### END CUP



FOR SIZE	CODE
1.50 - 2.50 mm	571.0301
3.00 - 4.00 mm	571.0302

#### PORTLAND ELASTIC NAIL (TITANIUM)



DIAMETER (Ø)	LENGTH	CODE
1.50	300 mm	571.0416
2.00	400 mm	571.0401
2.50	400 mm	571.0402
3.00	400 mm	571.0403
3.50	400 mm	571.0404
4.00	400 mm	571.0405

#### CHOOSING NAIL DIAMETER

Size (Ø) of Medullary Canal	Age of Patient	Ø Nancy Nail
5,0 - 6,0 mm	4,0 - 6,0 Years	Ø 2,0 mm
6,0 - 7,5 mm	4,0 - 6,0 Years	Ø 2,5 mm
7,5 - 8,5 mm	6,0 - 8,0 Years	Ø 3,0 mm
8,5 - 10,0 mm	9,0 - 11,0 Years	Ø 3,5 mm
10,00 mm +	12,0 - 14,0 Years	Ø 4,0 mm


**Note :** The Nancy nail is not intended for use in children under 14 years of age or in patients over 65 Kg Nail diameter selection depends on both age and medullary canal diameter.

567.0000

**PORTLAND TITANIUM ELASTIC NAIL COMPLETE SET**

567.0004

TRAY

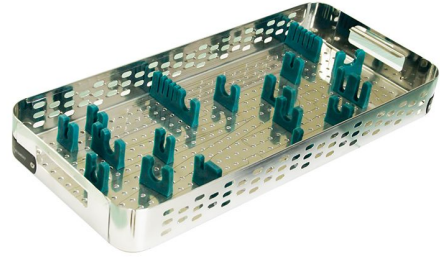

568.0125

INSERTER



568.0308

HAMMER GUIDE



568.0300

ROD GRIP



568.0410

COMBINED HAMMER



568.0111

STRAIGHT IMPACTOR



568.0112

BEVELLED IMPACTOR



568.0115

STRAIGHT AWL



568.0120

CURVED AWL



568.0700

1.5 mm. - 2.5 mm. END CUP

568.0800

3 mm. - 4 mm. END CUP



568.0400

STRAIGHT CUTTER



**Hipokrat Tıbbi Malzemeler İmalat ve Pazarlama A.Ş.**

7407/6 Sokak No:10 35060 Pınarbaşı Bornova - İZMİR / TÜRKİYE

+(90) 232 479 11 09

+(90) 232 479 44 29

info@hipokrat.com.tr

www.hipokrat.com.tr



**HIPOKRAT**